
ventura **unibond 2**

Ⓔ [Solvent-free 1-component dental adhesive](#)

Ⓔ [Adhesivo dental monocomponente sin disolventes](#)

Madespa
www.madespa.com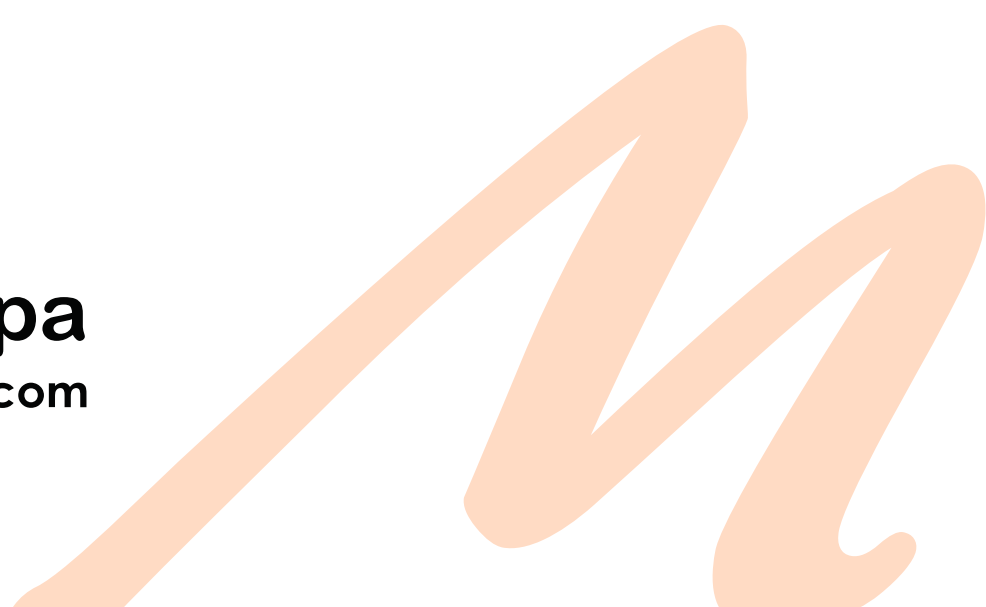

ventura **unibond 2**

Solvent-free 1-component dental adhesive

EN

INSTRUCTIONS FOR USE

MD

1. DEFINITION

Ventura Unibond 2 is a light-curing, solvent-free 1-component restorative adhesive in the total-etch process for enamel and dentine.

2. COMPOSITION

Ventura Unibond 2 contains 99% of methacrylate-based monomers (including HEMA derivatives) and 1% of catalysts and inhibitors.

3. INTENDED USE / INDICATIONS FOR USE

- Direct restorations of all cavity classes on anterior and posterior teeth with light-curing composites.
- Direct restorations and core build-ups with dual-curing or self-curing composites.
- Adhesive bonding of indirect restorations (crowns, bridges, inlays, onlays, veneers) with light-curing, dual-curing or self-curing composite cements.

The clinical benefit of this product for the patient is the restoration and maintenance of tooth function and aesthetics through reliable adhesive restorative procedures. It enables adhesive restorative procedures by promoting bonding between dental hard tissues and composite-based restorative materials.

4. CONTRAINDICATIONS

Exposed pulp, pulpitis, known allergy to methacrylate resin.

5. INTENDED USERS

This product is intended exclusively for use by duly qualified dental professionals.

6. INTENDED PATIENT POPULATION

Patients who require the application of a dental adhesive to promote adhesion between dental tissue and restorative materials in restorative or preventive dental procedures. There is no available data related to special patient populations such as children and during pregnancy or breastfeeding. Therefore, the use of the dental adhesive is subject to the criteria and professional judgement of the health professional.

7. SIDE EFFECTS

Isolated cases of contact-allergies with products of similar composition have been reported. To avoid possible pulpal reactions, protect the exposed dentine in the pulpal area with an appropriate lining material (preferably with a calcium hydroxide cavity liner).

8. INTERACTIONS

Avoid underfilling materials which may hinder polymerization owing to their ingredients. All phenolic compounds, such as zinc oxide, eugenol or preparations containing thymol, belong to this category.

9. INSTRUCTIONS FOR USE

9.1. Drying

Due to the following application of the filling material, a drainage of the working field is required. Avoid any contamination of the cavity with blood or saliva. A rubber dam is recommended.

9.2. Cavity preparation

Prepare the cavity as usual. Undercuts and bevelled margins are recommended to improve adhesion and the margin fit. Clean and dry the cavity. Avoid over drying.

9.3. Etching of enamel and dentine

Apply the etching gel and condition in correspondence to instruction for use of the product. Sufficiently etched enamel appears matt and white.

Note: Possibly use the total-etch-technique in cavities limited by dentine. In this case, the dentine should not be etched for more than 15 seconds and only dried briefly after rinsing.

9.4. Application

Apply to enamel/dentine in a not too thin layer with a brush and massage into the tooth substance for 20 seconds.

Note: **Ventura Unibond 2** is light-curing, so avoid too intense ambient light. Dim the surgical light during the application.

9.5. Drying

Dry the adhesive layer with an air blower for at least 5 seconds.

9.6. Polymerizing

Polymerize the bonding layer with blue light (halogen or LED lamp) for 30 seconds. The specified exposure time relates to halogen or LED light-curing devices with a minimum light intensity of 500 mW/cm² and a wavelength of 400 - 500 nm.

Note: Do not use **Ventura Unibond 2** for opaque restorations or deep cavities where complete light-curing is not guaranteed.

Application and processing of the composite filling material according to the instructions for use of the product used.

10. EXPIRY

The shelf life of **Ventura Unibond 2** is up to three years. Do not use after the expiry date.

11. MARKING OF THE PACKAGING

The lot number and expiry date are printed on each container.

12. STORAGE

Close **Ventura Unibond 2** bottle tightly after use. Store in an upright position to ensure the backflow of the liquid. Do not expose light-curing products to direct sunlight or operating light. **Ventura Unibond 2** was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Constant temperatures above 28°C / 82°F can reduce the shelf-life of the product.

13. DISPOSAL

The product must be disposed of in accordance with applicable local regulations. Do not discharge the liquid product into drains or dispose of it with household waste.

14. PRECAUTIONS

Do not use the product if the container is damaged, leaking or shows signs of deterioration. Close the bottle properly after each use. Keep material out of the reach of children. Commercially available medical gloves do not provide protection against the sensitization effect of methacrylates. If the product comes into contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

15. EMERGENCY PROCEDURES

In case of direct contact with the oral mucous, rinse with plenty of water. In case of contact with the eyes, rinse thoroughly with water. In case of doubt consult an ophthalmologist.

16. MULTIDOSE USE AND HYGIENE

Ventura Unibond 2 is supplied in a multidose bottle intended for use in multiple patients by a dental professional. The bottle does not come into direct contact with the patient and is not intended to be reprocessed, cleaned, disinfected or re-sterilized after clinical use. Dispense the material in a controlled manner. Use application instruments for one patient only. Do not allow the bottle tip to come into contact with oral tissues or contaminated instruments. If contamination of the bottle is suspected, the product must not be used further.

17. ADDITIONAL NOTES

These materials have been developed to be used exclusively by dentists and must be handled according to the instructions for use. The recommendations included in the instructions correspond to the current state of knowledge regarding dental techniques. The manufacturer cannot be held responsible for damages caused by other uses or incorrect handling.

Caution: Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

18. EXPLANATION OF SYMBOLS ON LABELLING

	Used-by date		Do not use if package is damaged and consults instructions for use
	Manufacturer		Medical device
	Consult electronic instructions for use		Catalogue no.
	Batch code		Caution
	Temperature limit		Model no.
	Keep dry		Unique device identifier
	Keep away from sunlight		

Instructions for use issued in July 2026 (2026-07)

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MADESPA, S.A.