ventura top 85 lab

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1. DESCRIPCIÓN

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EN INSTRUCTIONS FOR USE

1. DESCRIPTION

VENTURA TOP LAB 85 is a polysiloxane to be mixed with a gel catalyst, characterised by good final hardness (>80 Shore A), high stability, excellent detail reproduction and resistance to deformation. Suitable for extra-oral use only.

The intended uses of **VENTURA TOP LAB 85** are:

- Moulds for the fabrication of temporary prosthetic restorations; addition or replacement of elements in complete or framework dentures.
- Removable gingival moulds.
- Partial counter-moulds in flasks for partial or complete dentures.
- Blocking during denture repairs.
- Occlusal registration keys for positioning in an articulator.
- Duplication of complete dentures and gypsum models without the use of separating agents.
- Support for the fabrication of light-curing composite resins.

2. PREPARATION

Dosage:

Using the measuring scoop, take the required amount of **VENTURA TOP LAB 85** and spread it over the palm of the hand (note: the measuring scoop must be level).

Press the edge of the measuring scoop into the material as many times as the number of scoops dispensed.

For each scoop, apply one strip of **CATALYST FOR LABORATORY CONDENSATION SILICONE** of the same length as the measuring scoop (equivalent to 2.5% of the weight of the base).

Mixing/use:

Fold the material over itself and knead vigorously with the fingertips for approximately 30 seconds until a uniform colour without streaks is obtained.

At this point, begin shaping the material according to the intended application.

The working time, including mixing and using the gel catalyst at room temperature, is approximately 2–3 minutes.

The setting time is approximately 5–6 minutes.

3. TECHNICAL CHARACTERISTICS

	Ventura Top Lab 85
Mixing time	30"
Working time	2'00 – 3'00"
Setting time	5'00 – 6'00"
Shore A Hardness	85
Deformation under compression	≤ 1,5%
Linear dimensional change	- 0, 10%

4. WARNINGS/PRECAUTIONS

The indicated times are measured from the start of mixing at 23°C (73°F). High temperatures or an excessive amount of catalyst accelerate setting, whereas low temperatures or an insufficient amount of catalyst increase the setting time. Avoid contact with clothing. The use of gloves is recommended. Avoid contact of the catalyst with the skin and eyes. In the event of accidental skin contact, wash thoroughly with soap and water. In the event of contact with the eyes, rinse thoroughly with running water and seek medical advice. If swallowed, contact a doctor immediately.

5. STORAGE

Do not store at temperatures above 25°C (77°F). Protect from direct sunlight and do not use after the expiry date. Close the containers tightly after use.

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