ventura impress 2.0 putty soft

REFERENCE	DESCRIPTION
3390	VENTURA IMPRESS 2.0 PUTTY SOFT FAST SET
3392	VENTURA IMPRESS 2.0 PUTTY SOFT REGULAR SET

ES

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1. DESCRIPCIÓN

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INSTRUCTIONS FOR USE

1. DESCRIPTION

Ventura Impress 2.0 is a system of materials for high precision dental impressions based on addition curing silicones (polyvinylsiloxane), particularly indicated for the double impression technique. Both the mouldable putty, **Ventura Impress 2.0 Putty Soft**, as well as the fluid paste **Ventura Impress 2.0 Light Body**, packaged in cartridges, possess, once polymerized, excellent mechanical properties and a great capacity for reproducing details. Handling is quite simple, since in all cases the base putty and the catalyst have the same density, and the mixture is always made with a 1:1 ratio (by weight or by volume). The different colours of the two pastes provide good control of the mixture. The use of the fluid paste in cartridges greatly simplifies the procedure, producing homogeneous mixtures with a total absence of bubbles and a notable economy in material usage. The hydro-compatibility of **Ventura Impress 2.0 Putty Soft** allows it to enter into close contact with humid surfaces without absorbing liquid, making possible a more perfect reproduction of details, particularly on the edges of the preparation. This property also makes possible a better gypsum flow over the sur- face when preparing the model, penetrating in even the smallest cracks. The polymerization of the polyvinylsiloxane does not produce volatile secondary components. This gives it excellent dimensional stability, which permits the preparation of the model even several days after taking the impression, without clinically notable alterations.

2. INDICATIONS

Very high viscosity impression material recommended for use as a tray material in single step (sandwich) or double step techniques.

3. CONTRAINDICATIONS

It should not be used in patients with allergy to the components of the device.

4. INTENDED PATIENT POPULATION

Patients undergoing a dental procedure, with no limitation regarding age or gender, except where the product is contraindicated due to known hypersensitivity or allergy to any of its ingredients.

5. INTENDED USE

Dental treatments that require an impression of the denture of the patient.

6. PREPARATION

Draw equal amounts of the putty and the catalyst paste from their respective containers, using the corresponding measuring spoons for each component. With fingertips, mix both components until a smooth paste with uniform colour is obtained (approximately 30 seconds). Use vinyl gloves, because the powder that protects most latex gloves can inhibit polymerization of the silicone. Close the containers immediately following their use. Make sure to replace the correct covers and measuring spoons for each container. If the ingredients are handled separately before mixing, the setting time will be shorter.

7. MAKING THE IMPRESSION

Once the mixture of **Ventura Impress 2.0 Putty Soft** is completed, place the mixed material in the tray (can be metal or plastic, with or without perforations). Within 1 minute from starting the mix, place the tray with the material in the mouth and keep it in position for at least 1 minute and 30 seconds (fast set) or 2 minutes and 30 seconds (regular set).

8. MODELS

Models can be poured after 30 minutes, and within seven days of polymerization. The impression should be kept at room temperature until the moment of removal.

9. ELECTROPLATING (GALVANIZATION)

Ventura Impress 2.0 Putty Soft is compatible with all quality gypsums, such as **Fuji Rock, Whip Mix** or **Ventura**.

9. ELECTROPLATING (GALVANIZATION)

Ventura Impress 2.0 Putty Soft impressions can be galvanized by acid baths of copper or silver alkaline.

10. PRECAUTIONS

In people sensitive to polyvinylsiloxane, these can cause irritation or other allergic reactions. Contact with latex gloves may prevent vinylpolyisiloxane from setting. Avoid direct or indirect contact with latex gloves. Instead, wash your hands thoroughly to eliminate all traces of impurities or use vinyl gloves. Silicone adheres to clothes from where it may be very difficult to remove, even by using chemical agents.

11. STORAGE

In order for the product to maintain its optimum characteristics, it is advisable to store it at temperatures between 5 °C and 25 °C and use it before the expiry date.

12. TECHNICAL CHARACTERISTICS

	Putty Soft Regular Set	Putty Soft Fast Set
Type	Mouldable putty	
Classification ISO 4823	Very high viscosity - Type 0	
Colour	Yellow	Blue
Mixing ratio base:catalyst	1 : 1	
Mixing time	30 seconds	
Working time at 23 °C	1 min 45 sec.	1 minute 15 sec.
Setting time	3 minutes	2 minutes
Compression deformation	4 % approx.	
Recovery following deformation	≥ 99%	
Linear dimensional variation at 24 hours	≤ 0,5%	
Shore A Hardness at 24 hours	68 approx.	

13.DISPOSAL

Dispose of the product in accordance with applicable local regulations.

14. FOR PROFESSIONAL USE ONLY

These materials have been developed to be used exclusively by dental professionals and must be handled according to the instructions for use. The recommendations included in the instructions correspond to the current state of knowledge regarding dental techniques. The manufacturer cannot be held responsible for damages caused by other uses or incorrect handling.

15. REPORTING OBLIGATIONS

Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

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