ventura crystalsil

REFERENCE	DESCRIPTION
3275	VENTURA CRYSTALSIL (2x50 ml) + 12 MIXING TIPS

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INSTRUCTIONS FOR USE

1. DESCRIPTION
Ventura Crystalsil is a transparent material based on vinylpolisiloxanes suitable for all types of bite registrations that require transparent matrices.

2. INDICATIONS
Accurate occlusal registrations.
Key material for needle point registration.
Production of small model segments.
Silicone key for composite restorations.

3. CONTRAINDICATIONS
It should not be used in patients with allergy to the components of the device.

4. INTENDED PATIENT POPULATION
Patients undergoing a dental procedure, with no limitation regarding age or gender, except where the product is contraindicated due to known hypersensitivity or allergy to any of its ingredients.

5. INTENDED USE
Dental treatments requiring bite registrations with transparent matrices.

6. DIRECTION FOR USE
Insert cartridge onto dispensing gun following the manufacturer's instructions and remove cap. Extrude impression material until it exits both chambers at the same rate and then mount the appropriate mixing tip. An intra-oral tip may be placed on the mixing tip.

7. APPLICATION
Apply Ventura Crystalsil onto a suitable transparent impression tray, keeping the end of the mixing tip immersed in the material to avoid the formation of air bubbles, and place it in the mouth. Carefully remove the set matrix after the recommended time. Rinse with water and air-dry. Prepare the teeth according to the desired technique and in accordance with the composite material manufacturer's instructions for use. Apply the composite onto the transparent matrix and position it in the mouth. Light-cure through the matrix following the corresponding instructions for use. After use, keep the mixing tip on the cartridge, as it serves as a cap. Intraoral tips and mixing tips are for single use only.

8. DISINFECTING
Once the bite registration is removed from mouth, rinse under cold water. It can then be disinfected with appropriate disinfecting solutions or sprays.

9. PRECAUTIONS
In people sensitive to polyvinylsiloxane, these can cause irritation or other allergic reactions. Contact with latex gloves may prevent vinylpolisiloxane from setting. Avoid direct or indirect contact with latex gloves. Instead, wash your hands thoroughly to eliminate all traces of impurities or use vinyl gloves. Silicone adheres to clothes from where it may be very difficult to remove, even by using chemical agents.

10. STORAGE
In order for the product to maintain its optimum characteristics, it is advisable to store it at temperatures between 10 °C and 25 °C (50 – 77 °F) and use it before the expiry date.
Store bite registration at room temperature (max. 25°C / 77 °F).

11. LOT NUMBER AND EXPIRY DATE
Lot number and expiry date are printed on each package and should be quoted in all correspondence which requires identification of the product.

12. TECHNICAL CHARACTERISTICS

	Crystalsil
Classification ISO 4823	Type B
Colour	Transparent
Mixing time	Automixing
Total working time	1 min (*)
Time in oral cavity	2 min
Total setting time	3 min (*)
Linear dimensional variation at 24 hours	< 0,3 %
Shore A Hardness at 24 hours	70 ± 2

(*) Application is at 23 °C ± 2 / 73 ± 4 °F and 50 ± 5% relative humidity. Higher temperatures reduce working and setting times and lower temperatures prolong them.

13. DISPOSAL
Dispose of the product in accordance with applicable local regulations.

14. FOR PROFESSIONAL USE ONLY
These materials have been developed to be used exclusively by dental professionals and must be handled according to the instructions for use. The recommendations included in the instructions correspond to the current state of knowledge regarding dental techniques. The manufacturer cannot be held responsible for damages caused by other uses or incorrect handling.

15. REPORTING OBLIGATIONS
Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

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