ventura bite 2.0

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INSTRUCTIONS FOR USE

1. DESCRIPTION
Ventura Bite 2.0 is a silicone elastomer of the vinylpolysiloxane type whose reticulation takes place based on a polyaddition reaction. The reticulated product's mechanical properties and reproduction of detail are excellent. **Ventura Bite 2.0** presents a high thixotropy and low contraction. It does not adhere to organic resins or gypsum.

2. INDICATIONS
Accurate occlusal registrations.
Key material for needle point registration.
Production of small model segments.
For applications that require hard bite registrations.

Ventura Bite 2.0 is compatible with Ventura Stone (Type III) and Ventura Superdie Rock (Type IV).

3. CONTRAINDICATIONS
It should not be used in patients with allergy to the components of the device.

4. INTENDED PATIENT POPULATION
Patients undergoing a dental procedure, with no limitation regarding age or gender, except where the product is contraindicated due to known hypersensitivity or allergy to any of its ingredients.

5. INTENDED USE
Dental treatments that require a bite registration of the patient.

6. PREPARATION OF THE DISPENSING GUN
1. Load the cartridge by sliding it vertically through the leads provided for this purpose
2. Press the lever until the plunger rests on the cartridge.

7. MIXING
Unscrew the cartridge cap by turning it 90° (1/4 of a revolution), remove it and discard it, as it cannot be reused. Extrude impression material until the same amount comes out from the cartridges bores and then mount the appropriate mixing tip, blocking it by means of a 90° turn (1/4 of a revolution). For a more precise dispensing, a bite registration oral tip may be placed on the mixing tip.

8. MAKING THE IMPRESSION
Apply bite registration material according to the desired technique (occlusal or buccal). Extrude material first onto areas where teeth have been prepared keeping the end of the mixing tip immersed in the material to avoid the formation of air bubbles and ask the patient to bite. The material should be placed directly on the teeth within 30 seconds from starting the mix and kept in occlusion position for at least 45 seconds. Carefully remove the set bite registration after the recommended time. Remove all residues of the material from the mouth. Trim bite registration to ensure correct seating on model. Once you have finished the procedure, leave the mixing tip on the cartridge until the next use as it serves as a cap. Between uses, store the dispensing gun with the mixing tip facing down. Intraoral tips and mixing tips are for single use only.
For the next mix using the same cartridge, remove the used mixing tip by turning it 90° (1/4 of a revolution), insert a new mixing tip and proceed.

9. DISINFECTING
Once bite registration is removed from mouth, rinse under cold water. The impression can then be disinfected with appropriate disinfecting solutions or sprays such as 2% glutaraldehyde. Take care to follow the manufacturer's instructions for use.

10. PRECAUTIONS
In people sensitive to polyvinylsiloxane, these can cause irritation or other allergic reactions. Contact with latex gloves may prevent vinylpolysiloxane from setting. Avoid direct or indirect contact with latex gloves. Instead, wash your hands thoroughly to eliminate all traces of impurities or use vinyl gloves. Silicone adheres to clothes from where it may be very difficult to remove, even by using chemical agents.

11. STORAGE
In order for the product to maintain its optimum characteristics, it is advisable to store it at temperatures between 5 °C and 25 °C and use it before the expiry date.

12. LOT NUMBER AND EXPIRY DATE
They are printed on each package.

13. TECHNICAL CHARACTERISTICS

	Bite 2.0
Classification ISO 4823	Type B
Colour	Lavender
Mixing ratio base:catalyst	1 : 1
Mixing time	Automixing
Working time at 23 °C	12 s (*)
Setting time	45 s (*)
Linear dimensional variation at 24 hours	≤ 0,5%
Shore A Hardness at 24 hours	89 approx.

(*) These times are intended from the beginning of mixing at 23 °C (73 °F). Higher temperatures shorten while lower temperatures prolong these setting times.

14.DISPOSAL
Dispose of the product in accordance with applicable local regulations.

15. FOR PROFESSIONAL USE ONLY
These materials have been developed to be used exclusively by dental professionals and must be handled according to the instructions for use. The recommendations included in the instructions correspond to the current state of knowledge regarding dental techniques. The manufacturer cannot be held responsible for damages caused by other uses or incorrect handling.

16. REPORTING OBLIGATIONS
Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

REFERENCE	DESCRIPTION
3396	VENTURA BITE 2.0 (2x50ml)

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