ventura bulkfill

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EN 1. PRODUCT DESCRIPTION

Ventura bulkfill flow flowable, light-curing and X-ray opaque microhybrid composite with very low shrinkage and an increased curing depth (increment thickness 4mm), which meets the requirements of ISO 4049. When using as a basis for filling, **ventura bulkfill flow** has to be coated occlusally with an at least 2 mm thick layer of a methacrylate-based universal or lateral tooth composite. **Ventura bulkfill flow** is used together with a dental adhesive. The total filler volume is 26,00 % and the particle size is of 0.7 - 2.6 microns.

2. COMPOSITION

Barium glass, silanised, ytterbium trifluoride BisGMA, BisEMA, catalysts, inhibitors, additives.

3. INDICATION

- Lining (basis for filling) and lining for class I and II cavities.
- Direct filling therapy of small class I dental restorations in remaining molars and for class I and II dental restorations in primary molars (without the need for an additional occlusal composite material).
- Extended fissure sealing.
- Core build-ups.

4. CONTRAINDICATION

Exposed pulp, pulpitis, known allergy to methacrylate, camphorquinone and BHT.

5. SIDE EFFECTS

Contact allergies to products of a similar composition have been described in isolated cases. To prevent pulp reactions, it is advisable to cover exposed dentine in the region of the pulp with suitable liners (preferably with a calcium hydroxide). **Note: ventura bulkfill flow** contains no TEGDMA and no HEMA.

6. INTERACTIONS

Avoid using liners that may impair polymerisation based on their ingredients. This group includes all phenolic compounds, such as zinc oxide eugenol or thymol-containing preparations.

7. APPLICATION

7.1 Drying

Dry sufficiently, apply dental rubber dam.

7.2 Preparing the cavity

Prepare the cavity according to the principles of the adhesive filling technique (minimally invasive). Then clean and dry the cavity.

7.3 Placing the matrices

Take the necessary precautions in areas which **ventura bulkfill flow** can flow into. Using matrices that are wedged into the approximal area can be helpful. Minimal separation facilitates the approximal contact and the insertion of the matrix.

7.4 Lining

For preparations near the pulp, it is advisable to protect the pulp by means of a lining (e.g. calcium hydroxide cement). Do not use zinc oxide eugenol!

7.5 Adhesion

Prior to inserting the filling, it is necessary to apply an adequate adhesive system to the enamel and dentine according to the manufacturer's instructions. We recommend **ventura unionbond 2**.

7.6 Application

Syringe: Fix the included dispensing tip onto the syringe and directly apply **ventura bulkfill flow**.

Starting with the lowest point, directly insert the material into the prepared cavity and fill the cavity from the bottom to the top. Leave the tip of the metal tube inserted in the cavity. To prevent bubbles from forming, ensure a slow and even flow of the material. If necessary, remove bubbles and excess material prior to polymerisation. When using as a basis for filling, the distance to the occlusal edge of the cavity must be at least 2 mm. **Ventura bulkfill flow** can be polymerised in increments of up to 4 mm thick.

7.7 Light-curing

Use standard polymerisation devices for light-curing. Sufficiently expose every area of the filling. Additionally exposing the accessible approximal box and the oral and vestibular areas is also recommended. If using metal matrices, these must be removed beforehand. Bring the light window of the lamp as close as possible to the surface of the filling (otherwise the curing quality will be poorer). Insufficient curing can lead to discolouration of the material, to loss of strength of the composite filling and to complaints. The exposure time is 20 seconds (for halogen lamps with 700 mW/cm²). The exposure time may vary depending on the source of light and the corresponding instructions for use.

7.8 Occlusal coating

When using as a basis for filling, els bulkfill has to be coated with an at least 2 mm thick layer of a methacrylate-based universal or lateral tooth composite. We recommend the composite **ventura similux** or **ventura nanolux**. Regarding the processing and elaboration, please note the respective instructions for use.

8. WARNINGS/PRECAUTIONS

- Bring the composite material to room temperature before use.
- The included dispensing tips are for single use only and have to be disposed after each use. Replace the cap onto the syringe after each use.
- After termination of the application, do not retract the piston. Instead, turn the piston slightly to relieve the pressure so as to avoid any unnecessary leakage of the substance.
- Always use **ventura bulkfill flow** tips for one patient only!
- Commercially available medical gloves do not provide protection against the sensitisation effect of methacrylates. If the product comes into contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

9. STORAGE

Protect light-curing products from strong sources of light and heat! Recommended storage conditions: 4 – 28 °C. In case of room temperatures above 28 °C, storage of the products in the refrigerator is recommended. Do not freeze!

10. LOT NUMBER AND EXPIRY DATE

The lot number is used for identification of the product in case of enquiries. Do not use the products after the expiry date.

11. PRECAUTIONS

Close containers with the correct lid after each use. For dental use only. Keep out of the reach of children.

12. EMERGENCY MEASURES

In case of direct contact with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an ophthalmologist.

13. HYGIENE

Always use application instruments for one patient only. Clean and disinfect multiple-use instruments after use. Dispense products at an adequate distance from the patient to avoid contamination.

14. WARRANTY

Our liability is limited to the quality of our products. If the quality of a product is faulty, only the value of the product will be replaced. No liability will be accepted for other damages, in particular damages caused by non-compliance with the instructions for use or other incorrect handling or improper use of a product. Before using the products, it is the responsibility of the user to check whether the products are suitable for the intended purpose. The user expressly accepts all risks associated with the use of the product and bears the sole responsibility for any arising damages.

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MADESPA, S.A. - Río Torvisgal, 8 - 45007 TOLEDO - ESPAÑA (SPAIN)