# ventura unibond 2

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### 1. DEFINICIÓN

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## EN INSTRUCTIONS FOR USE

### 1. DEFINITION

**Ventura Unibond 2** is a light-curing, solvent-free 1-component restorative adhesive in the total-etch process for enamel and dentine.

### 2. COMPOSITION

**Ventura Unibond 2** contains :

Hydroxyethylmethacrylate  
Uretanemethacrylate  
Polyalkenoate methacrylized  
Hydroxypropylmethacrylate  
Glycerinedimethacrylate  
Catalyst, inhibitors

### 3. INDICATION

- Direct restorations of all cavity classes on anterior and posterior teeth with lightcuring composites.
- Direct restorations and core build-ups with dual-curing or self-curing composites.
- Adhesive bonding of indirect restorations (crowns, bridges, inlays, onlays, veneers) with light-curing, dual-curing or self-curing composite cements.

### 4. CONTRAINDICATIONS

Exposed pulp, pulpitis, known allergy to methacrylate resin.

### 5. SIDE EFFECTS

Isolated cases of contact-allergies with products of similar composition have been reported. To avoid possible pulp reactions, protect the exposed dentine in the pulpal area with an appropriate lining material (preferably with a calciumhydroxide cavity liner).

### 6. INTERACTIONS

Avoid underfilling materials which may hinder polymerization owing to their ingredients. All phenolic compounds, such as zinc oxide eugenol or preparations containing thymol, belong to this category.

### 7. APPLICATION

#### 7.1. Drying

Due to the following application of the filling material, a drainage of the working field is required. Avoid any contamination of the cavity with blood or saliva. A rubber dam is recommended.

#### 7.2. Cavity preparation

Prepare the cavity as usual. Undercuts and bevelled margins are recommended to improve adhesion and the margin fit. Clean and dry the cavity. Avoid overdrilling.

#### 7.3. Etching of enamel and dentine

Apply the etching gel and condition in correspondence to instruction for use of the product. Sufficiently etched enamel appears matt and white. Note: Possibly use the total-etch-technique in cavities limited by dentine. In this case, the dentine should not be etched for more than 15 seconds and only dried briefly after rinsing.

#### 7.4. Application

Apply to enamel/dentine in a not too thin layer with a brush and massage into the tooth substance for 20 seconds. Note: **Ventura Unibond 2** is light-curing, so avoid too intense ambient light. Dim the surgical light during the application.

#### 7.5. Drying

Dry the adhesive layer with an air blower for at least 5 seconds.

#### 7.6. Polymerizing

Polymerize the bonding layer with blue light (halogen or LED lamp) for 30 seconds. The specified exposure time relates to halogen or LED light-curing devices with a minimum light intensity of 500 mW/cm<sup>2</sup> and a wavelength of 400 - 500 nm. Note: Do not use **Ventura Unibond 2** for opaque restorations or deep cavities where complete lightcuring is not guaranteed. Application and processing of the composite filling material according to the instructions for use of the product used.

### 8. BATCH NUMBER AND EXPIRY DATE

The lot number and expiry date are printed on each container. Products should no longer be used once the expiry date has elapsed.

### 9. STORAGE

Close **Ventura Unibond 2** bottle tightly after use. Store in an upright position to ensure the backflow of the liquid. Do not expose light-curing products to direct sunlight or operating light. **Ventura Unibond 2** was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Constant temperatures above 28°C / 82°F can reduce the shelf-life of the product.

### 10. PRECAUTIONS

Close the jar properly after each use. Keep material out of the reach of children. Commercially available medical gloves do not provide protection against the sensitisation effect of methacrylates. If the product comes into contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

### 11. EMERGENCY MEASURES

In case of direct contact with the oral mucous, rinse with plenty of water. In case of contact with the eyes, rinse thoroughly with water. In case of doubt consult an ophthalmologist.

### 12. HYGIENE

Use application instruments for one patient only. Clean and disinfect other instruments after each use. Dispense the material away from the patient to avoid contamination.

### 13. ADDITIONAL NOTES

The material has been developed for exclusive use by dental professionals.

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MADESPA, S.A. - Río Torviscal, 8 - 45007 TOLEDO - ESPAÑA (SPAIN)