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DESCRIPCIÓN

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INSTRUCTIONS FOR USE

DESCRIPTION

Ventura Seal is designed for sealing the enamel pits and fissures of teeth to aid in prevention of caries. Ventura Seal is a light curing single component and fluoride containing pitand fissure sealant. The white colouration allows controlled application and facilitates visual ob- servation.

COMPOSITION

- Dimethacrylates
- Dental glass
- Photo initiators

INDICATIONS/INTENDED USE

Sealing of fissures and pits and prevention of caries.

PERFORMANCE FEATURES

The performance features of the product meet the requirements of the intended use.

CONTRAINDICATIONS

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

PATIENT TARGET GROUP

Persons who are treated during a dental procedure

INTENDED USERS

This medical device should only be used by a professionally trained dental practitioner.

INCOMPATIBILITY WITH OTHER MATERIALS

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

APPLICATION OF VENTURA SEAL

Rubber dam is the recommended method of isolation.

1. Enamel Etching

Ventura Etching Gel roughens the enamel surface and produces microfine retentions to enable Ventura Seal to flow properly. Thus, the etching of the enamel increases the adhesion of Ventura Seal in fissures, pits and on the tooth surface. The occlusal surfaces of the teeth to be sealed must be thoroughly clea- ned. Apply Ventura Etching Gel in the surface area and keep the acid off from dentin. Leave it in place for 30 seconds and flush then thoroughly with water. Dry with oil and water free air. The etched enamel should have a chalky white appearance. Etching precaution: IT IS ESSENTIAL THAT THE ETCHED ENAMEL IS NOT CONTAMINATED BY ANYTHING. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, rinse immediately with copious amounts of water. During the application of Ventura Seal, the areas should be kept dry with rubber dam.

2. Application of sealant

2.1. Direct application with needle tips: Attach the needle tip to the syringe. Press cautiously Ventura Seal in a thin layer directly onto etched areas and – if necessary - spread with a micro- brush. 2.2. Application with a microbrush: Place a small quantity of Ventura Seal onto the tooth and spread with a microbrush. Ventura Seal should only be applied to the etched areas. Re- move excess of material. Light cure each segment with a LED or halogen lamp (approx. 470 nm) for 20 seconds.

3. Sealant evaluation

Inspect the sealant for complete coverage and voids. If surface has not been contaminated, additional sealant may be added. If contamination has occurred, re-etch, rinse, and dry prior to placing more sealant.

4. Dismissal

Wipe the sealant with a cotton applicator to remove the thin sticky film onthe surface. Check occlusion and adjust as required.

ADDITIONAL NOTES / WARNINGS:

- Ventura Seal contains methacrylates. Avoid use of this product on patients with known methacrylates allergies.
- Avoid use on patients with allergies to any of its components.
- The ambient light of the dental lamp may start polymerization of the material.
- Do not use any resin to adjust viscosity of the material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized material may have an irritant effect and may cause sensitization or allergic reaction against methacrylates in sensitive individuals.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!
- For dental use only.

STORAGE:

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

DISPOSAL

Disposal of the product according to local authority regulations.

REPORTING OBLIGATION

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

NOTE

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

WARRANTY

First Scientific Dental GmbH warrants this product will be free from defectsin material and manufacture. First Scientific Dental GmbH makes no other warranties, including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, his exclu-sively remedy and First Scientific Dental GmbH sole obligation shall be thereplacement of First Scientific Dental GmbH product.

LIMITATION OF LIABILITY

Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, inci-dental or consequential, regardless of the theory asserted, including wa-ranty, contract, negligence or strict liability.

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