ventura dupligel

ES. INSTRUCCIONES DE USO

DESCRIPCIÓN

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EN. INSTRUCTIONS FOR USE

DESCRIPTION

Ventura DUPLIGEL it's a gelatine duplicating material agar-agar based, suitable for investments.

AVANTAGES

- It presents an optimal hardness, elasticity and stability to achieve duplicates of precision and with good definition of details.
- Its great elasticity makes possible to achieve an extremely smooth model surface.
- No bubbles during hardening
- Reduction of costs for their use continuous and bigger output.
- Easy to use.

HANDLING GUIDELINES

• PREPARATION

- Cut the gelatin into cubes (about 1 cm on the edge).
- Heat up in a bain-marie or in a gelatin machine to a temperature of 90°C without exceeding it.
- For use in microwave: heat approx. 500 g. of VENTURA DUPLIGEL 4 minutes at 800W power, stir carefully and return to heat 1 minute at 400W.
- Leave cool the product to room temperature up to 40-50°C (recommended use temperature), to which the product can be poured into the flask. To speed up this process, we can introduce the container with the gelatin in a larger one with cold water. Stirring this way provides uniform cooling.
- Hydrate the master model (if it is gypsum) to facilitate its removal from the mold. By immersing it completely in water for approximately 15 minutes.
- Let it rest for approximately 30 minutes (gelation time) in the case of a metal flask. In case of using plastic flask, wait for approximately 45 minutes.

DUPLICATING

- At the end of the hardening time, carefully remove the master mold.
- Wash the piece with water and drain well (if there are traces of water or dirt, the surface of the duplicate may be rough).

- Then fill with the coating and vibrate for 2 minutes. Let the model dry in the open air to room temperature.

- CAUTION: Too cold gelatin can lead to a longer setting time and a weaker, rougher duplicate.

RE-USE

VENTURA DUPLIGEL is designed for more than one use. the following indications should be followed:

- Wash the used gelatine carefully removing all investment rests
- Cut into small pieces and melt as described in the preparation section.
- If the gelatin is very dense, small amounts of distilled water can be added to compensate for the loss of water due to evaporation.

The product is reusable until it is observed that it loses its original characteristics (fluidity, plasticity, color). When this occurs, discard and use new gelatin.

STORAGE

Keep the product in his original package and in a cool and dry place.

PHYSICAL PROPERTIES

Melting point: 90 °C. (194°F)

Solidification point: 38 °C app. (82°F)

Temperature of use: 40-50 °C (115°F)

Gelification time: 35 min.

PRESENTATION

2420 – VENTURA ENDUMODEL
BUCKET 6 kg.