

ventura endumodel

Endurecedor de modelos

Model hardener

5.000 ml

LOT 2421-xxxx



XXXX-XX



MODO DE EMPLEO:

Para modelos duplicados en revestimiento para esqueléticos.

Sumergir el modelo caliente una vez secado (según instrucciones del fabricante del revestimiento) en un recipiente con VENTURA ENDUMODEL hasta cubrirlo totalmente y mantenerlo sumergido durante 10 segundos (repetir otros 3 segundos si se desea una capa más gruesa/dura). Para facilitar la posterior adaptación de la cera sobre el modelo, se recomienda aplicar una o dos capas de VENTURA ENDUMODEL con un pincel, una vez el modelo se haya enfriado completamente.

USE:

For investment duplicated models in Cr-Co technique.

Immerse the hot model once it has dried (according to the investment manufacturer's instructions) in a container with VENTURA ENDUMODEL until completely covered and kept submerged for 10 seconds (repeat for another 3 seconds if a thicker / harder coat is desired). To facilitate the subsequent adaptation of the wax on the model, it is recommended to apply one or two coats of VENTURA ENDUMODEL with a brush, once the model has completely cooled down.



MADESPA, S.A - Río Jarama, 120 -
45007 - TOLEDO - ESPAÑA (SPAIN)
www.madespa.com

