ventura **3lux**
naturally

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3LUX teeth are manufactured with double cross linked DLC and high molecular weight Methylmethacrylate Polymer, within the most modern techniques and under rigid quality control. All production processes comply with ISO 13485 and they are certified Biocompatible in accordance to ISO 10993-5.

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SYNTHETIC POLYMER TEETH FOR DENTAL PROSTHESES

Follow the usual procedures for choice, waxing-up, articulation, polymerization, infection-control and maintenance of the prosthesis. For an optimized bonding to the denture base resin, grind the tooth base with a grinding instrument. After insulating the flask and prior to pack the acrylic dough, clean the base surface of the tooth using methylmethacrylate monomer.

Indications for the patient: to increase the teeth durability, it is highly recommended to avoid contact with alkalis acids or other substances that can damage acrylic materials during the daily hygiene.

DISTRIBUIDO POR / DISTRIBUTED BY:
MADESPA, S.A.
RÍO TORVISCAL 8, 45007 - TOLEDO (SPAIN)
Tel: (+34) 925 241 025
info@malespa.com
www.malespa.com

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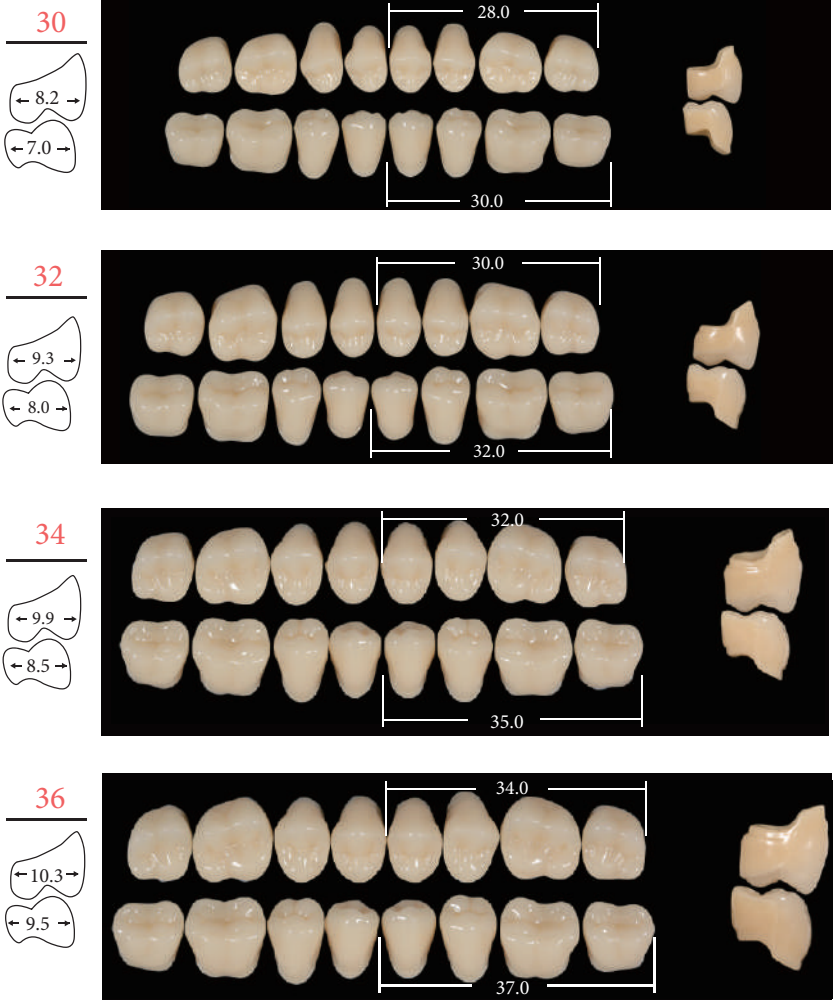
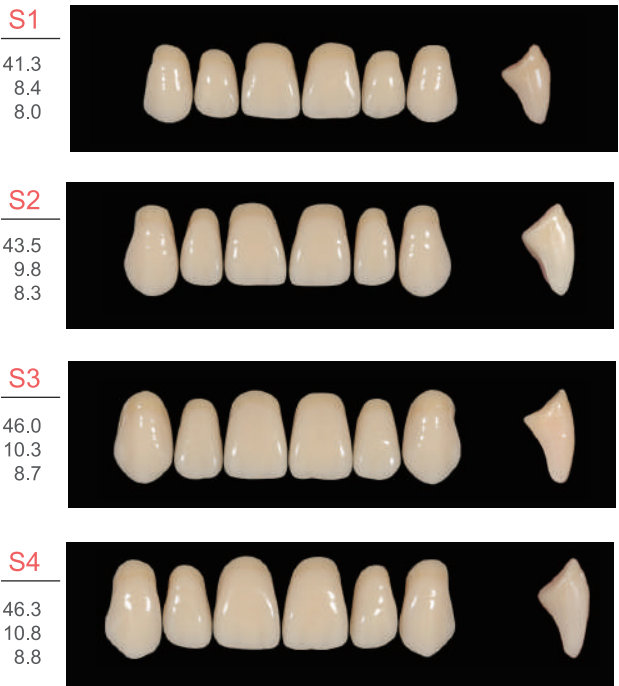
Posteriors with thicker enamel design and better wear resistance

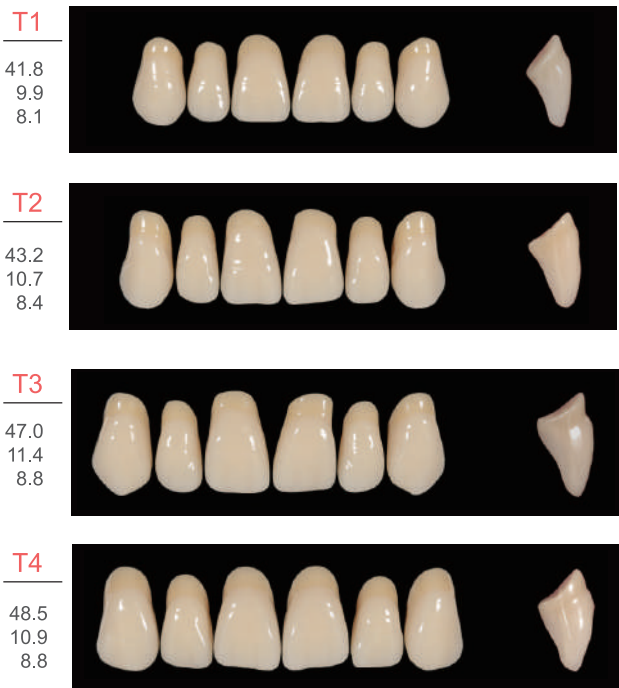
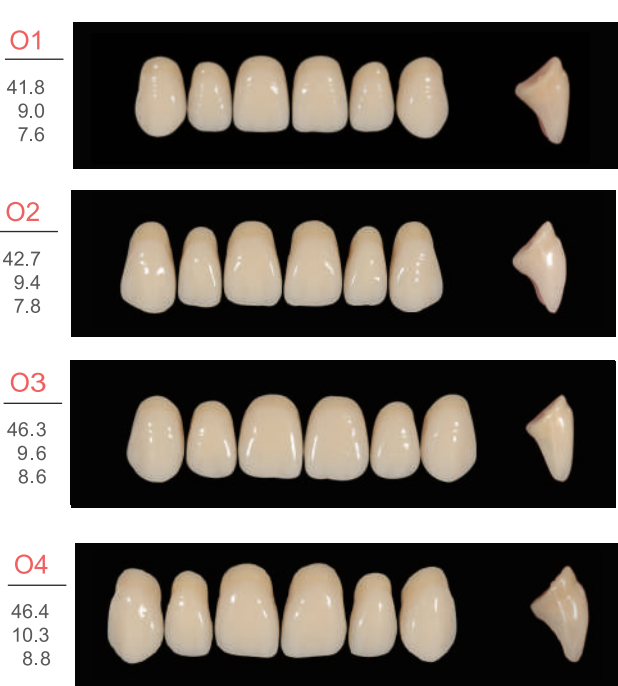
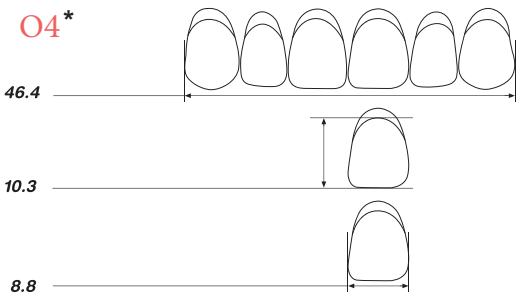
Fine powder can be obtained when enamel layer is grinded, which stands for higher wear resistance. Thicker enamel layer of VENTURA 3LUX posterior maintains tooth function after grinding and leads to better wear-resistance and anti-stain ability.

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